
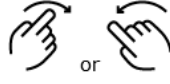

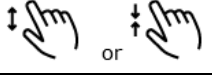
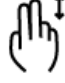




Mobile Device Use

Multi-touch application gestures include:

Scroll through images	Drag up or down 
Move to the next or previous image	Flick right or left 
Display images in full-screen mode -or- Exit full-screen mode	Double-tap 
Zoom an image	Stretch or pinch 
Pan an image	Two-finger drag 
Adjust the W/L of an image	Window: three-finger drag right or left  Level: three-finger drag up or down 

Change Healthcare Enterprise Viewer Physician Quick Reference Guide



**Need Help?
Password reset?**
Contact your Help Desk



Image Link via EMR

Images are available via an icon in the EMR when accessing the patient's report. Click the **Image Link** icon to display the patient's images for that exam.

Accessing from the Desktop



Double-click the **Enterprise Viewer** icon on the desktop. Use your domain username and password to access.

Patient Search options include Last Name, First Name and Patient ID, Ordering System and Date of Birth.

Patient Search Study Search

Search Patient

Last Name Patient ID Date of Birth mm/dd/yyyy

First Name Ordering System All


Study Search allows you to locate a study by Accession Number.

Patient Search Study Search

Search Studies

Accession


Close Patient or Logout

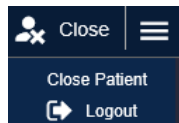

Click **Close** and select **Close Patient** to close the current patient you are viewing. To exit the application, select **Logout**.

Work Area


View the patient's studies on the timeline

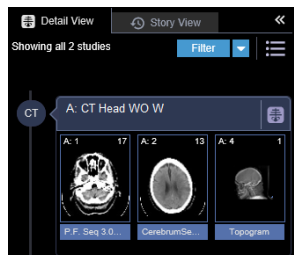

Select the **Filter** drop-down to narrow the list of studies displayed. The filters available are determined by the patient's studies.

Story View

Study Cards display each study's key images and reports.

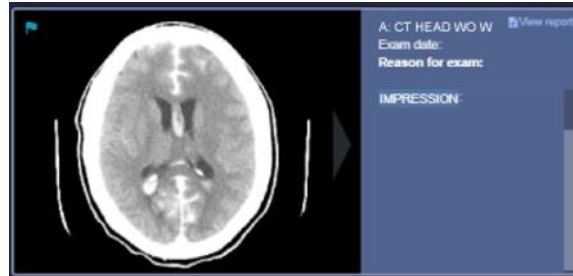

Double-click the image to display in full-screen mode. Click the arrows to move to the next or previous key image.

Detail View

Display a patient's full set of study images, view patient documentation and compare studies.

Scroll through Images: Series within CT and MRI may be scrolled. Left-click and roll the mouse wheel to scroll through the images.



Power Scroll through Images: Click the Power Scroll icon and then left mouse drag.



Click the **Image and Report Layout** to view the images and reports or scanned documents together.




Click the **Image Layout** on the top portion of the screen.



Choose the viewports layout. Drag an image or series from the timeline to any viewport.

Right Click Menu

Right-click an image or series to view these available options (Annotations are temporary):

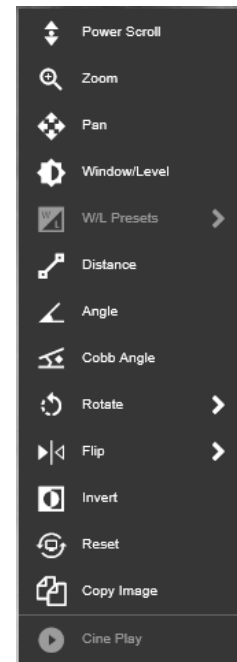

Power Scroll: Allows you to scroll/navigate quickly.

Zoom: Allows for interactive zoom.

Pan: Allows you to navigate zoomed images.

Window/Level: Allows for interactive window/leveling left click and drag.

Window/Level Presets: Applies predefined window/level values.

Distance: Allows you to measure.

Angle: Measures the angle between two intersecting lines.

Cobb Angle: Allows you to assess the angle between two non-intersecting lines.

Rotate: Reorient all images in a series clockwise or counterclockwise.

Flip: Reorient all images in a series by flipping them horizontally or vertically.

Invert: Allows you to reverse the gray values of all images in a series.

Reset: Reset images in a series to their original displayed state, before any image manipulations.

Copy Image: Copy images from the Viewer to other applications

Cine Play: Display images as an animated sequence of frames.